# NORSE®

## Disposable Respirator Masks FFP2 & FFP3 NR D

EN 149:2001+A1:2009 CE 2797

NORSE Denmark ApS, Smedevej 16C, 7430 Ikast, Denmark  
EU Declarations of conformity available for download at: [www.norseshop.com](http://www.norseshop.com)

### EN INSTRUCTIONS FOR USE Disposable respirator mask with valve

CE CAT. III

#### READ CAREFULLY:

Single-use respirator masks are certified according to the European PPE Regulation EU 2016/425 on safety requirements for personal protective equipment and are classified according to one of three categories: FFP1, FFP2 or FFP3. Before reading the following documentation, check which category the mask belongs to - this can be found on the packaging and mask. It is the user's responsibility to ensure that the mask used provides the necessary protection against the type and concentration of contaminant(s) in the environment where the mask is intended to be used.

**'NR' means the masks are for single-use only**

**'D' means the masks have meets the Dolomite clogging test**

#### DIRECTIONS FOR USE:

- Fit the mask and check for tightness BEFORE entering the pre-cleaned area
- Wear the mask the entire time you are exposed to the contamination
- Use the mask according to the applicable regulations

Replace the mask with a new one if:

- The mask is removed in a contaminated area
- Severe clogging of the filter makes breathing difficult or uncomfortable
- The mask is damaged
- (for masks that protect against vapours) - If vapours can be smelled inside the mask

Leave the contaminated area if dizziness, irritation or other discomfort occurs. The mask is for single use only and therefore maintenance is not required.

The masks must not be stored after use or recycled. Discard the mask after use (maximum one use). Store unused masks in the unopened packaging in a dry and clean place between -20°C and +40°C at a relative humidity below 75%.

**NOTIFIED BODY:** BSI Group Netherlands B.V. (2797), Say Building, John M. Keynesplein 9, 1066 EP Amsterdam, The Netherlands. The Notified Body conducted the Regulation Type-Examination and are responsible for the ongoing quality surveillance.

#### FITTING DIRECTIONS:

- Hold the mask in your hand, let the head straps hang down freely.
- Hold the mask under your chin with the nose piece facing outwards
- Pull the bottom strap over your head and place it at the nape of your neck so that it is under your ears. Hold the mask against your face with one hand, pull the top strap over your head and position the top of the back of your head so that it is above your ears.
- For masks with adjustable straps: While holding the mask in place, adjust the straps by pulling the end of each band (loosen the band by pressing in on the buckle)
- Mould the nose area to the shape of your nose by pressing on the noseband.
- Check the mask for leaks (face fit): Place both hands over the mask and exhale hard. The overpressure should be noticeable inside the mask. If you can feel the air coming out, adjust the mask again by squeezing the nose clip and/or adjusting the straps. Repeat the procedure until the correct fit is achieved.

#### LIMITATIONS:

Do not use these masks or stay in areas where:

- Oxygen levels are below 17% (UK 19%)
- There are unknown air pollutants or concentrations that are immediately hazardous to health or life
- The particle concentration exceeds the approved limit values or the MAC x NPF limit values (Nominal Protection Factor: 4.5 for FFP1, 12.5 for FFP2 and 50 for FFP3)
  - NPF: Nominal protection factor
  - MAC: Maximum authorised limit value
- There are gases or vapours - except if the respirator is intended as protection against gases and vapours, in which case the concentration must not exceed the limit value. Respirators intended for protection against organic vapours cannot be used for acid vapours and vice versa.
- FFP2 NR: Filter efficiency 94%: The assigned protection factor (APF) is 10
- FFP3 NR: Filter efficiency 99%: The assigned protection factor (APF) is 20

#### WARNING:

Do not use for firefighting. These masks do not add oxygen. Do not use in oxygen-deficient areas such as tanks or poorly ventilated areas (see restrictions). Do not use in explosive environments. Use only by trained personnel. If the mask is worn over a beard or stubble, the correct seal is unlikely to be achieved. When transporting the product, keep it in its original packaging and keep it away from mechanical and chemical influences.

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